

Bioline™ H.PYLORI Ag

H.PYLORI ANTIGEN TEST

Bioline™ H.pylori Ag kit is a rapid, qualitative test for the detection of *Helicobacter pylori* antigen in human fecal specimens.

- Easy to use
- No need of any equipment
- Test result: 10 -15 minutes
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 98.4 %, Specificity 100 % (vs. respiratory test and CLO test)
- Specimen: Fecal specimens



MATERIALS PROVIDED

- Test device
- Specimen collection tube
- Assay diluent
- Specimen collection swab
- Disposable dropper
- Disposable dropping cap

SIMPLE PROCEDURE

1 Add Assay Diluent
Transfer assay diluent twice.

2 Insert swab specimen
Insert the swab into the specimen collection tube and swirl the swab at least 10 times.

3 Assemble dropping cap
Assemble dropping cap on the specimen collection tube.

4 Add Specimen
Dispense 3 drops.

RESULTS INTERPRETATION

NEGATIVE

C T

POSITIVE

C T

INVALID

C T

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
H.pylori Ag	04FK20	Device	Fecal	20T/Kit