

Bioline™ H.PYLORI

H.PYLORI ANTIBODY TEST

Bioline™ H.pylori test is a rapid test for the qualitative detection of antibodies of all isotypes (IgG, IgM, IgA, etc.) specific to *Helicobacter pylori* in human serum or plasma.

- Detection of all isotypes (IgG, IgM, IgA) antibodies against *H.pylori*
- Shelf life and storage temperature: 24 months from the date of manufacturing at 2-30 °C
- Performance: Sensitivity 95.9 %, Specificity 89.6 % (vs. ELISA)
- Specimen : Serum, Plasma

MATERIALS PROVIDED


- Test device/Multi-device
- Assay diluent



SIMPLE PROCEDURE


1 Add Specimen

Dispense 10 µl of serum or plasma into the specimen well "S".

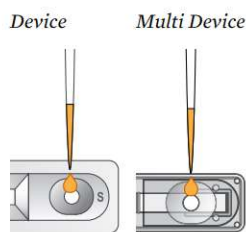

2 Add Assay Diluent

Dispense 3 drops of the assay diluent.

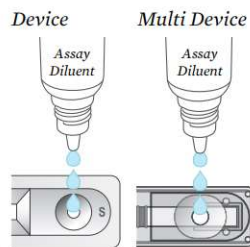

Wait 10 mins.



RESULTS INTERPRETATION

NEGATIVE

POSITIVE

INVALID

INVALID

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
H.pylori	04FK10	Device	Serum/Plasma	30T/Kit
H.pylori	04FK11	Multi-Device	Serum/Plasma	10Tx10/Kit