

# Bioline™ HCV


## HEPATITIS C VIRUS ANTIBODY TEST

Bioline™ HCV test is a immunochromatographic rapid test for the qualitative detection of antibodies specific to HCV in human serum, plasma or whole blood.

- Recombinant HCV Core, NS3, NS4, NS5 Ag used as capture materials
- Specimen: Serum, Plasma, Whole blood
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 99.3 %, Specificity 98.1 %

### MATERIALS PROVIDED

- Test device / Multi-device / Strip
- Assay diluent
- Option : Lancet, alcohol swab, capillary pipette (for fingerstick)

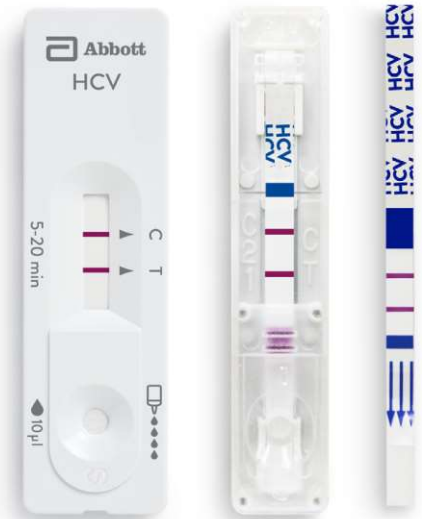

### SIMPLE PROCEDURE

**1 Add Specimen**  
 Dispense 10 µl of specimen into the specimen well.

Capillary pipette    Micropipette    Multi Device

**2 Add Assay Diluent**  
 Dispense 4 drops of the assay diluent.

Device    Multi Device

**1 Add Assay Diluent and Specimen**  
 Dispense 4 drops of assay diluent to the empty test tube, and then dispense 10µl of serum or plasma to the test tube.

Strip

**2 Insert Strip**  
 Insert strip into the test tube.

Strip



**RESULTS INTERPRETATION**

**NEGATIVE**

**POSITIVE**

**INVALID**

The presence of any test line, no matter how faint, the result is considered positive.

### ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
HCV	02FK10	Device	Serum/Plasma/Whole blood	30T/Kit
HCV	02FK11	Multi-Device	Serum/Plasma/Whole blood	10Tx10/Kit
HCV*	02FK16	Device	Serum/Plasma/Whole blood	25T/Kit
HCV**	02FK17	Device	Serum/Plasma/Whole blood	25T/Kit
HCV	02FK10CE	Device	Serum/Plasma/Whole blood	30T/Kit
HCV*	02FK16CE	Device	Serum/Plasma/Whole blood	25T/Kit
HCV**	02FK17CE	Device	Serum/Plasma/Whole blood	25T/Kit
HCV Fast	02FK12	Strip	Serum/Plasma	25T/Kit

(\*) Lancet, Capillary pipette, Alcohol swab included.  
 (\*\*) Safety lancet, Capillary pipette, Alcohol swab included.