

# Bioline™ INFLUENZA Ag A/B/A (H1N1)

## INFLUENZA VIRUS TYPE A, B AND A (H1N1) PANDEMIC RAPID TEST

Bioline™ Influenza Ag A/B/A (H1N1) rapid test kit is a chromatographic immunoassay for the differential and qualitative detection of Influenza virus type A, type B and A (H1N1) antigens directly from nasal/throat/nasopharyngeal swab or nasal/nasopharyngeal aspirate specimens.

- Detection: Differential detection of Influenza virus type A, type B and A (H1N1) antigen (4 lines)
- Specification: The samples tested POSITIVE can be determined as influenza H1N1 positive without additional confirmation test.
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C




### MATERIALS PROVIDED


- Test strip
- Disposable tube with rack
- Assay diluent
- Sterilized swab
- Control swab: A positive, B positive, Negative
- Disposable dropper

### SIMPLE PROCEDURE


#### All swab specimens

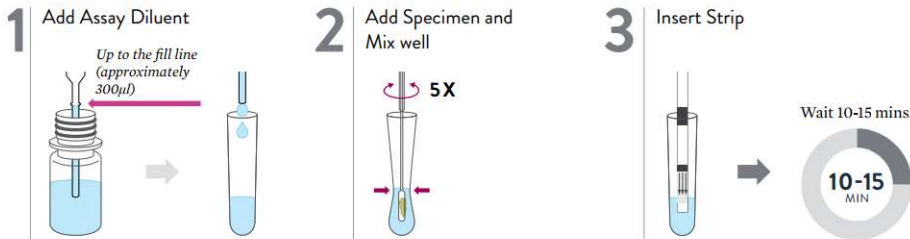

#### Nasal aspirate, nasopharyngeal aspirate specimen or extracted specimen into transport media

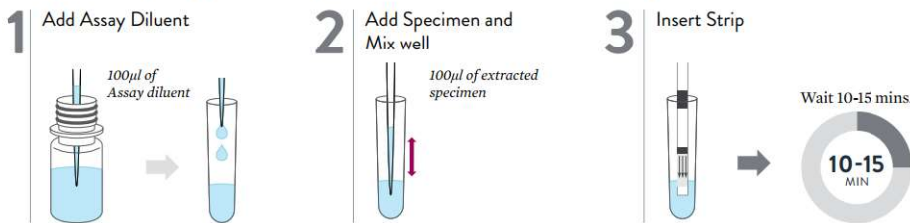

#### RESULTS INTERPRETATION

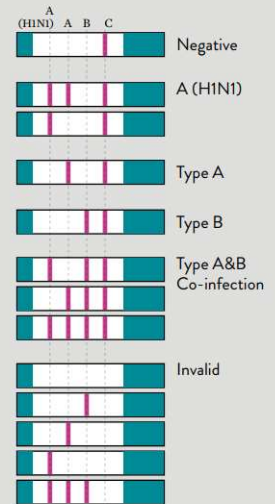

The presence of any test line, no matter how faint, the result is considered positive.

### ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Influenza Ag A/B/A (H1N1)	19FK31	Strip	Nasal/Throat/Nasopharyngeal swab, Nasal/Nasopharyngeal aspirate	10T/Kit
Influenza Ag A/B/A (H1N1)	19FK32	Strip	Nasal/Throat/Nasopharyngeal swab, Nasal/Nasopharyngeal aspirate	25T/Kit