

Bioline™ INFLUENZA ANTIGEN

INFLUENZA VIRUS TYPE A & B ANTIGEN TEST

Bioline™ Influenza Antigen test is a chromatographic immunoassay for the differential and qualitative detection of influenza virus type A and type B antigens directly from nasal / throat / nasopharyngeal swab or nasal/nasopharyngeal aspirate specimens.

- Detection : Differential detection of Influenza virus type A & B
- Specimen : Human nasal swab, throat swab, nasopharyngeal swab or nasal/nasopharyngeal aspirate
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity : 91.8%, Specificity : 98.9% (vs. Viral Culture and RT-PCR as gold standard)




MATERIALS PROVIDED


- Test strip
- Disposable tube with rack
- Assay diluent
- Sterilized swab
- Control swabs: Positive and Negative
- Disposable dropper

SIMPLE PROCEDURE

All swab specimens

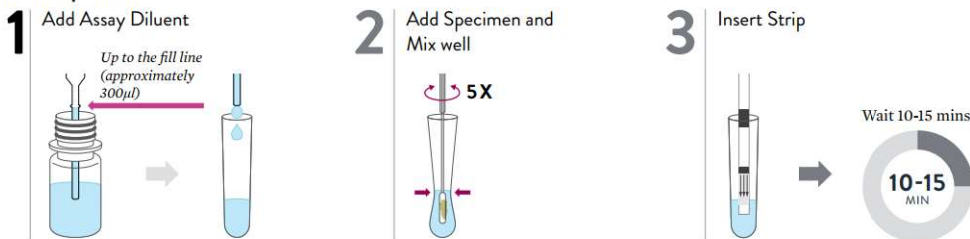

Nasal aspirate, nasopharyngeal aspirate specimen or extracted specimen into transport media

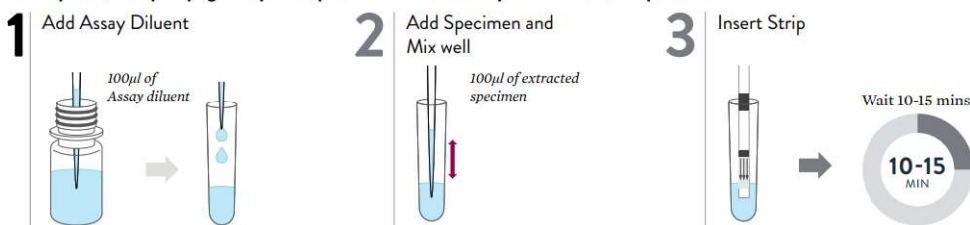

RESULTS INTERPRETATION

NEGATIVE




Type A-POSITIVE



Type B-POSITIVE



INVALID

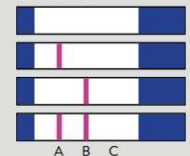

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Influenza Antigen	19FK11	Strip	Nasal/Throat/Nasopharyngeal swab, Nasal/Nasopharyngeal aspirate	10T/Kit
Influenza Antigen	19FK12	Strip	Nasal/Throat/Nasopharyngeal swab, Nasal/Nasopharyngeal aspirate	25T/Kit