

Bioline™ LEISHMANIA Ab

LEISHMANIA ANTIBODY TEST

The Bioline™ Leishmania Ab test kit is rapid immunochromatographic assay for *in vitro* diagnostic use designed to detect *Leishmania* antibodies in human serum or plasma.

- Easy to use rapid test : All MATERIALS PROVIDED
- Specimen : Serum, plasma (20 µl)
- Test result : 10-15 minutes
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 98.0 %, Specificity 99.5 % (vs. immune-fluorescent assay)


MATERIALS PROVIDED

- Test strip
- Assay diluent
- Capillary pipette (20 µl)
- Disposable test tube




SIMPLE PROCEDURE


1 Add Assay Diluent
Dispense 3 drops of the assay diluent.

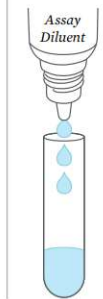

2 Add Specimen
Dispense 20 µl of serum or plasma

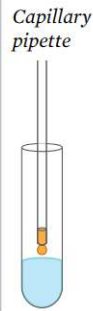

3 Insert Strip

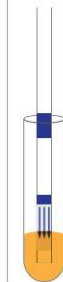

Wait 10-15 mins.




RESULTS INTERPRETATION

NEGATIVE



POSITIVE

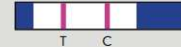

INVALID



The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Leishmania Ab	47FK12	Strip	Serum/Plasma	25T/Kit