

Dedicated to Delivery
Nasco
Life/form®



Airway Management Trainer Head LF03603U Instruction Manual



Life/form® Products by Nasco

About the Simulator

The **Life/form**® Airway Management Trainer Head is the most realistic simulator available for the training of intubation and other airway management skills. Nasco has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail. Landmarks include nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoid, false and true vocal cords, trachea, esophagus, lungs, and stomach.

Nasco's Airway Management Trainer Head allows you to practice oral, digital, and nasal intubation. E.T., E.O.A., PTL® , and Combitube® insertion can all be practiced as well. Suction techniques and proper cuff inflation can also be performed and evaluated.

With proper care, your **Life/form**® Airway Management Trainer Head will provide years of valuable service. Please review the instructions carefully for setup, cleanup, and maintenance procedures.

List of Components

- Airway Management Trainer Head
- 8-oz. Bottle of Lubricant

Lubrication

Before using the Airway Management Trainer Head, lubricate both the simulator and supplies being used with the Nasco lubricant provided. (See **figures 1 & 2.**)

Note: Nasco recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Airway Management Trainer Head. The use of a silicone-based lubricant may cause damage to the simulator, thus voiding Nasco's warranty on the trainer.



Figure 1



Figure 2



Figure 3

Attaching the Head to the Manikin

Nasco's Airway Management Trainer Head has the ability to be attached to both **CPARLENE**® and any of the Resusci® Anne™* brand CPR manikins.

To begin, turn the head face down and tip it upward slightly until the two keyways on the back of the neck slide under the ring and into the top of the torso. **(See figure 3.)** Now match the single keyway on the neck front up to the slot on the backside of the torso and push the head down until it snaps into place. Rotate the head back around so that it is facing up and in the correct position.



Figure 4

Connecting the Airways

Once the head is in place, connect the **CPARLENE's**[®] lung bag to the right bronchus by sliding the cap plug over the adapter. **(See figure 4.)**



Figure 5

Note: Our Airway Management Trainer Head will fit on all models of **CPARLENE**[®] and Resusci[®] Anne[™]* manikins. To adapt the Airway Management Trainer Head to the Resusci[®] Anne[™]*, we have included an adapter piece. **(See figure 5.)** Remove the Anne[™] head and the connected section of corrugated tubing down to the valve. Install the Airway Management Trainer Head and hook it

up to the lung system with the adapter piece. Place the red cap end over the white fitting of the right bronchus and hook the other end to the lung bag valve nipple (where the previous corrugated tubing had been connected).

To remove the head from the manikin, reverse the attachment procedure.



Figure 6

Cleaning and Maintenance

To clean the Airway Management Trainer Head, you will first need to remove the head from the manikin. Next, take the trainer to an area with a sink and open counter space. Using the red cap supplied, plug off the right bronchus (the esophagus and left bronchus should already be plugged). Stabilize the head on the counter face-up (towels work well for this) with the plugged tubes hanging over the sink. Carefully pour warm soapy water (a mild dish soap works best) into the mouth until the water level reaches halfway up the tongue. Now tilt the head back and bring the neck up 3" off the countertop. **(See figure 6.)**

***RESUSCI[®] ANNE[™] IS A TRADEMARK OF LAERDAL MEDICAL CORPORATION.**

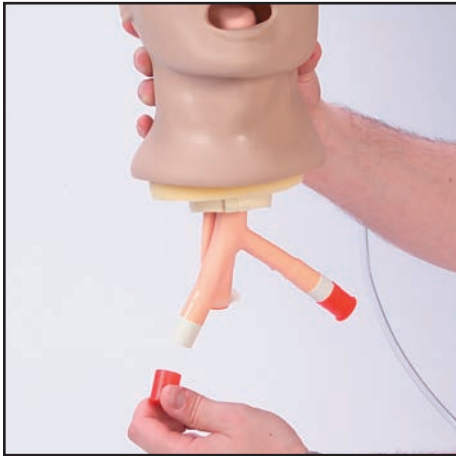


Figure 7

Continue filling until the water level covers the tongue. At this point, take a small soft brush and gently scrub the inside of the mouth. (A small toothbrush works well for this.) Cotton swabs can be used to scrub inside the nostrils. When done, pull the plug from the esophagus and drain into the sink. Now pick the head up, hold it in a vertical position, and pull the plugs from the trachea to completely empty the system. **(See figure 7.)**

Rinsing the Airway

To rinse the airway, follow the same procedure using clean, warm tap water. Repeat this process until all the soap has been flushed from the system.

Disinfecting the Airway

To disinfect, repeat the standard cleaning procedure, but this time use a bleach solution instead of soapy water, as specified by the Centers for Disease Control. Fill the system with the solution until it reaches the corners of the mouth. Remember to start filling with the head flat and finish with the neck slightly elevated to ensure that the solution completely fills all airway passages. Once completely filled with the bleach solution, allow the head to sit for at least 10 minutes. Drain as described earlier and repeat the rinsing process to flush out all of the bleach solution. Set the head aside and allow it to dry completely.

Cleaning the Face

Normal soil can be removed from the face of the trainer with mild, soapy water. Nasco Cleaner (LF09919U) will remove stubborn stains. Simply apply Nasco Cleaner to the soiled area and wipe clean with a soft cloth or paper towel. **Note:** Do not use Nasco Cleaner around the mouth and nostrils, as the residue of the cleaner could be toxic.

Caution

Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ball-point pens will also make indelible stains.

Supplies/Replacement Parts for the Airway Management Trainer Head

LF03644U Pump Spray Lubricant

LF09919U Nasco Cleaner

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00855U Male Catheterization
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00999U Pediatric Injectable Head
LF01005U First Aid Arm
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01034U Suture Practice Leg
LF01036U Spinal Injection
LF01037U Hemodialysis Practice Arm
LF01038U Episiotomy Suturing Set
LF01042U Suture Kit
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Infant Intraosseous Infusion
LF01121U Advanced IV Arm
LF01131U Venipuncture and Injection Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01143U Testicular Exam
LF01152U Male & Female Catheter
LF01155U Advanced CPR Dog
LF01162U Venatech IV Trainer
LF01174U NG Tube & Trach Skills
LF01184U Venatech IM & Sub Q
LF01193U Special Needs Baby
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer with Stand
LF03602U Adult Airway Management Manikin
LF03609U Child Airway Management Trainer with Stand
LF03616U Child **CRISis**™ Manikin
LF03617U Deluxe Child **CRISis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03623U Infant Airway Management Trainer with Stand
LF03632U Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer Torso
LF03693U **Basic Buddy**® CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03709U Infant **CRISis**™ Manikin
LF03720U **Baby Buddy**™ Infant CPR Manikin
LF03750U Fat Old Fred
LF03760U Airway Management/Cricoid Pressure Trainer
LF03770U Chest Tube
LF03953U **CRISis**™ Manikin, Complete
LF03955U Deluxe **CRISis**™ Manikin
LF03956U Deluxe "Plus" **CRISis**™ Manikin
LF03965U Adult **CRISis**™ Auscultation Manikin
LF03966U Adult **CRISis**™ Auscultation Manikin with ECG Simulator
LF04000U **GERI**™/**KERI**™ Manikin Series
LF04200U Adult Sternal Intraosseous Infusion
LF06001U CPR Prompt® Adult/Child Manikin
LF06012U CPR Prompt® Infant Manikin
LF06200U CPR Prompt® Keychain Rescue Aid
LF06204U CPR Prompt® Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1996 PRINTED IN U.S.A. NP 042-97/RV 7-12